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DESCRIPTION

TECHSUTURE PTFE Surgical Suture is a sterile, non-absorbable, monofilament synthetic surgical suture composed of 100% of polytetrafluoroethylene polymer. The PTFE suture is supplied in natural color (white).

INDICATION

TECHSUTURE PTFE Surgical Suture is indicated for approximation and ligation of soft tissue of the oral cavity, including membrane fixation and cuticular closure.

AVAILABILITY

TECHSUTURE PTFE Surgical Sutures are available in USP diameters from 7-0 to 1. They are supplied in natural color, sterile, pre-cut in a range of lengths, with or without swaged needles. TECHSUTURE PTFE Surgical Sutures are supplied in boxes of 6, 12, 24 and 36 units.

INSTRUCTIONS FOR USE

TECHSUTURE PTFE Surgical Sutures should be used according to the surgical procedure. The size and diameter of the suture thread and needle to be used in procedures will be established by medical and dental criteria. TECHSUTURE PTFE Surgical Sutures are not absorbable and should therefore be removed.

PRECAUTIONS

Like any other suture, care must be taken in its handling, avoiding damage, especially for being a monofilament.

Kneading, scraping or binding of the suture must be avoided when used in conjunction with surgical instruments such as needle driver and needle holders.

The needle should be grasped in the needle driver or needle holder between 1/3 and 1/2 distance between the swage end and the point of the needle.

Ensuring proper knotting requires surgical techniques of a flat knot followed by a square knot with several overlapping knots, indicated by the surgical circumstances and the surgeon's experience.

WARNINGS

The product should only be used by or under a qualified surgeon direction, according to the applicable surgical techniques and should only be handled by qualified personnel, under risk of wound dehiscence according to the application site and suture material used.

Do not use the product for purposes other than those recommended by the manufacturer. It is recommended that proper surgical practices be followed for the care of infected or contaminated wounds. Do not use product after its expiration date.

RESTRICTIONS

Reprocessing is forbidden.

Single-use health product.

Do not use if package is violated or wet.

Do not resterilize.

Discard the product opened and not used; discard suture leftovers.

SPECIAL CARE

Disposal of used sutures: Danger of biological contamination. Adopt disposal measures, considering the risk of biological contamination and perforating materials.

Disposal of unused sutures: Piercing-cutting material. Adopt disposal measures for piercing-cutting material.

When handling this product, care must be taken to ensure that it is handled aseptically.

Discard the used needles in proper recipients.

Care is recommended when handling surgical needles to avoid accidental punctures.

ADVERSE REACTIONS

Adverse reactions reported with the use of this product include wound dehiscence, minimal inflammatory tissue reaction and temporary irritation at the wound site and gradual loss of tensile strength over time.

CONTRAINDICATIONS

TECHSUTURE PTFE Surgical Suture is contraindicated for use in ophthalmic, neurological and cardiovascular surgery.

CONSERVATION

Transport and store at room temperature (15 to 30°C), in a cool, dry place, protected from humidity. Keep products away from direct sunlight, heat sources, dust and bad weather conditions.

SINGLE USE PRODUCT. DESTROY IT AFTER USE.

STERILE PRODUCT – ETHYLENE OXIDE.

Validity 5 years from date of manufacture.

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