---

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**DESCRIPTION**

TECHSUTURE Nylon Surgical Suture is a monofilament, formed by synthetic textile fibers based on polyamide resin, characterized by being elastic, non-absorbable, imputrescible and remarkably resistant to atmospheric agents.

**INDICATION**

Indicated for approximation of skin, soft tissues, and ligatures. Its use is also indicated for cuticular suture (cuticular closure). It is not indicated for cardiovascular and neurological surgeries.

The indicated time of permanence of the product in the patient is up to 30 days (short term).

**AVAILABILITY**

USP diameters from 11-0 to 3. They are supplied sterile, pre-cut in a range of lengths, with needles, in black, blue or colorless colors. Boxes of 6, 12, 24 or 36 units.

**INSTRUCTIONS FOR USE**

TECHSUTURE Nylon Surgical Suture should be used according to the surgical procedure.

Size and diameter of the suture thread and needle to be used in the procedures will be defined at the surgeon's discretion. Apply the suture on the edges of the surgical incision, using "interrupted" or "continuous" techniques according to the surgical techniques of incision closures and medical criteria.

TECHSUTURE Nylon Surgical Suture is not absorbed and must be removed.

**PRECAUTIONS, WARNINGS, SPECIAL CARE AND CLARIFICATIONS ON THE USE OF THE PRODUCT**

The product should only be used by or under the direction of a surgeon, according to applicable surgical techniques and should only be handled by qualified personnel.

When handling this product, care should be taken to ensure that it is done aseptically. Care should be taken in the handling of surgical needles to avoid accidental puncture.

Like any other suture, care must be taken in its handling, avoiding damage, especially for being a monofilament. Kneading, scraping or binding of the suture must be avoided when used in conjunction with surgical instruments such as needle drivers and needle holders.

The needle should be grasped with the needle driver or needle holder between 1/3 and 1/2 of the distance between the swage end and the point of the needle.

The guarantee of appropriate knots is given by the surgical circumstances and by the surgeon's experience, considering surgical techniques of flat knot followed by a square knot with several others superposed.

The time for the product to remain in the patient is of up to 30 days (short term).

Dispose of used needles in proper recipients.

Disposal of used sutures: danger of biological contamination, adopt disposal measures, considering the risk of biological contamination and perforating material.

Disposal of unused sutures: adopt disposal measures for piercing-cutting material.

**Do not use the product for purposes other than those recommended by the manufacturer.**

**Do not use the product after its expiration date.**

**Do not resterilize. Discard opened and unused product, discard leftovers.**

**Do not use if package is violated or wet.**

**Do not reprocess. Single-use health product.**

**CONTRAINDICATION**

Due to the gradual loss of tensile strength that can occur over an in vivo prolonged period, Nylon sutures should not be used if permanent retention in tensile strength is required.

**ADVERSE REACTION**

Mild erythema and pain at the suture site caused by surgical wound trauma to the tissues may occasionally occur. Adverse reactions associated with the use of this product include: wound dehiscence, gradual loss of tensile strength over time, stone formation in the biliary and urinary tracts when in prolonged contact with saline solutions such as bile and urine, minimal inflammatory tissue reaction and transient local irritation.

**CONSERVATION**

Transport and store at room temperature (15 to 30° C). Keep the product away from direct sunlight, free of dust, humidity and bad weather conditions.

---

**SINGLE USE PRODUCT. DESTROY IT AFTER USE.**

**STERILE PRODUCT - ETHYLENE OXIDE/ CO.60 GAMMA RADIATION**

**Validity 5 years from date of manufacture.**

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